


ThinPrep® 5000 processor and ThinPrep® 5000 Autoloader system


Quick reference guide

ThinPrep 5000 Processor

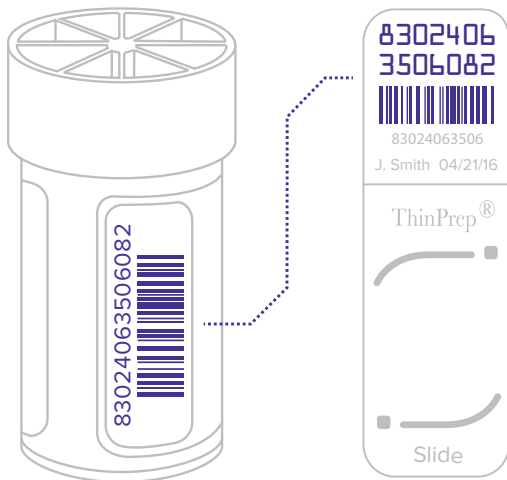

Vial and slide barcodes are matched.

ThinPrep 5000 Autoloader System

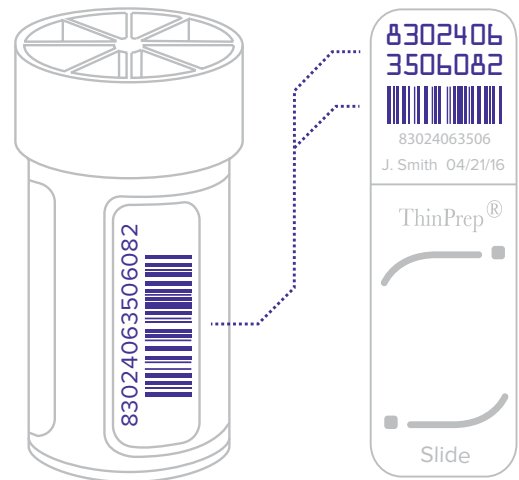

Vial barcode is read then barcode is etched on slide.

Note: ThinPrep® Imaging System users need to use the OCR format with the last 3 characters reserved as a CRC (CRC source code can be provided by the engineer).

Label Format

CODE 128	INTERLEAVED 2 OF 5	CODE 39	CODE 93	CODE 2D
Maximum of 8 alpha characters or 14 numeric digits	Only digits	Maximum of 6 characters	Maximum of 8 characters	Maximum of 64 characters

ThinPrep[®] 5000 processor and ThinPrep[®] 5000 Autoloader system

Quick reference guide

Label Specifications

VIAL BARCODE	SLIDE 1D BARCODE	SLIDE 2D BARCODE	SLIDE OCR

Third Party Vendor Labeling: Labels printed by third-party vendors can be used.

ThinPrep 5000 Autoloader mode:

- Slide information is laser etched directly onto the ThinPrep Slide.
- Users can determine the label format through the Slide Label Designer in the User Interface.
- Slide Label Designer guides the user through identifying information to be included on slide.
- Additional information such as date/time, instrument ID and LIS data can also be included on label.

The Slide Label Designer features include:

- Option to enable the Slide ID to be printed in OCR &/or user defined barcode.
- Option to print supplemental information in the upper and lower test regions.



hologic.com | diagnostic.solutions@hologic.com | +1.781.999.7300

DS-00121-001 Rev. 004 © 2016 Hologic, Inc. All rights reserved. Hologic, The Science of Sure, ThinPrep and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners. This information is intended for medical professionals in the U.S. and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative or write to diagnostic.solutions@hologic.com.

