


1 Prepare Speculum




- Warm water preferred
- Carbomer-free lubricant*

*If unavoidable, use minimal lubricant – no more than a **dime-sized** amount

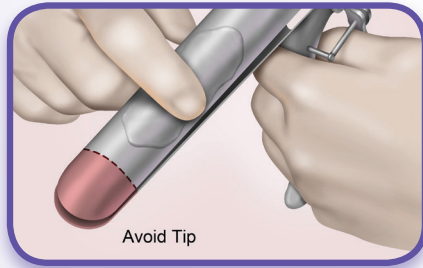

2 Obtain Sample

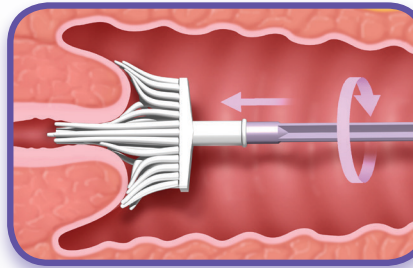

Rotate clockwise

360° 5x

3 Rinse Device



Rinse immediately

Push into bottom of vial forcefully **10 times** and swirl **vigorously**

DO NOT

leave device in vial

Hologic provides this quick reference guide ("guide") as a general informational tool only and is not an affirmative instruction or guarantee. While the information provided in this guide may describe a particular technique or protocol, it is not intended as an endorsement and/or requirement by Hologic to utilize such technique or protocol. It is the sole responsibility of the laboratory to determine which specific technique and/or protocol to employ. Laboratories using ThinPrep products must read and understand each product's Instructions for Use and comply with applicable local, state and federal rules and regulations.

MISC-03812-001 Rev. 001 © 2016 Hologic, Inc. All rights reserved. Hologic, The Science of Sure, ThinPrep and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your Hologic representative or write to diagnostic.solutions@hologic.com.